

F.A.N. PAD-GL™

Formaldehyde Absorbent Neutralizer Pad

“ULTRA-SHIPPER”

The Revolutionary Transport Insert Pad

**Ship & Receive Specimen Containers
with Confidence**

No More: Harmful Vapors

from leaking or damaged formalin prefilled specimen containers!



Simple to Ship: Simply Insert the 4 x 4 “F.A.N. Pad Ultra” and Specimen Container into a Bio Hazard Transport Bag. Seal and Ship

Easy to Use: Remove the Dry FAN Pad from the shipping bag.
Place a tissue cassette on the pad and pour the specimen into the cassette.
Place Larger Specimen on the F.A.N. PAD and Dissect.
F.A.N. Pad will Absorb & Neutralize the Formalin.

Reduces Formalin Vapor Exposure - **Reduces Transport Liability**
Reduces Risk of Lost Specimen - **Non-Hazardous Pad Disposal**

F.A.N. Pad GL™ “Ultra”
Safe – Simple – Effective

Product # FAN- 404 Packaged: 320 ea. 4” x 4” pads/ carton - 3 cartons/case (960-pads/case)
Each 4” x 4” Shipper Pad will absorb and neutralize up to 25-ml of 10% Formalin

American Bio-Safety, Inc.

4322 Anthony Court, # 5 Rocklin, CA. 95677

Bus: 916.652.8021 Email: absafety@sbcglobal.net Web Site: www.AmericanBioSafety.com Fax: 916.652.8020

F.A.N. PAD GL is a Registered Trademark of Sasco Chemical Group Inc.